

Easy User Guide

Mobil-O-Graph ABPM Without Integration

PREPARING THE DEVICE

1

Connect the Mobil-O-Graph **download cable** to your PC or laptop.



2

Connect the download cable to the **Mobil-O-Graph** and power on the device. **"CO"** should appear in the display window.



3

Open **HMS Client Server** from the desktop.



4

Click **add new patient** button.



5

Fill out the **form** and click **Save**.

Note: The minimum required fields are Patient ID, Name, First Name, Height, Weight, Date of Birth and Gender.


6

Click the **"Prepare Device"** button.



7

Once you have ensured all previous measurements have been downloaded from the device, click **"Delete Measurements."**

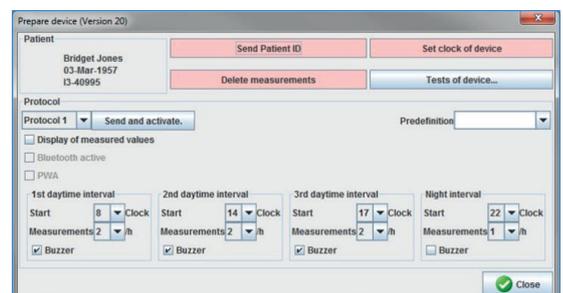

8

Click "**Set clock of device.**"

9

Click "**Send Patient ID.**"

10

Set the **first Daytime Interval** to when the patient is likely to **wake up** and the **Night Interval** to when the patient is **likely to go to bed.**

Note: Please note that the times are based on the 24hr clock and are hours only.

11

Set the "**Measurements**" per hour to the required number for each of the four intervals.

Note: 2 measurements per hour in the daytime and 1 measurement per hour at night is the norm.

12

Click "**Send and activate**" followed by **OK.**

13

Click "**Close.**"

14

You can now **exit** out of the software.



FITTING THE DEVICE TO YOUR PATIENT & THE FIRST MANUAL READING

1


Unplug the **Mobil-O-Graph** from the download cable and ensure that the **time** is displayed on the screen.

2

Place the **Mobil-O-Graph** in the pouch and fit the correct sized **cuff** to your patient.

3

You are now ready to take your first measurement, to start the manual measurement press the **Start / Stop** button located on the front of the device.

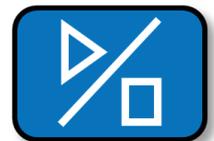

4

Once the **manual measurement** has been taken, your patient is ready to leave.

DOWNLOADING THE RESULTS

1

Connect the **Mobil-O-Graph** download cable to your PC or Laptop.



2

Connect the download cable to the **Mobil-O-Graph** and power on the device. **"CO"** should appear in the display window.



3

Open **HMS Client Server** from the desktop.




4

Click the **Patient List** button.



5

Locate your patient and click **Open patient**.

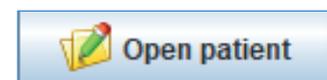


6

Click the **Read-out data** button.



7

Confirm the patient is correct on the **first box** and then click **Save** on the **second box** asking to confirm Measurement Series.

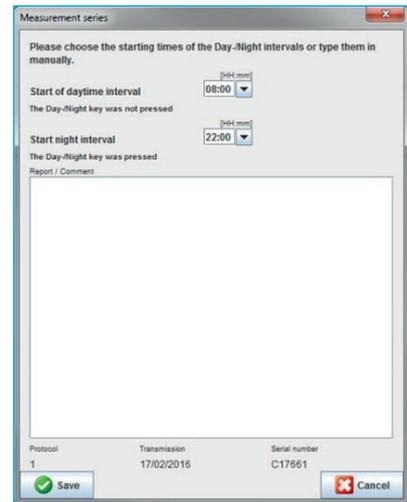

Measurement series

Please choose the starting times of the Day-Night intervals or type them in manually.

Start of daytime interval [hh:mm] 08:00
The Day-Night key was not pressed

Start night interval [hh:mm] 22:00
The Day-Night key was pressed


Report / Comment

Protocol: 1 Transmission: 17/02/2016 Serial number: C17661

Save Cancel

8

Once the results have been downloaded you will be asked if you want to remove the readings from the device.

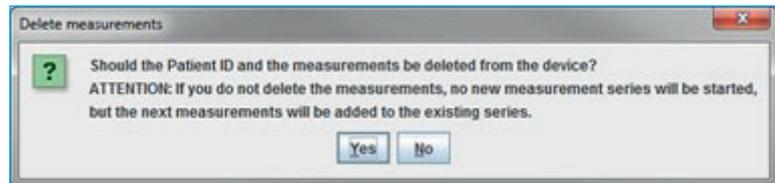

Delete measurements

? Should the Patient ID and the measurements be deleted from the device?
ATTENTION: If you do not delete the measurements, no new measurement series will be started, but the next measurements will be added to the existing series.

Yes No

9

Select the **print tab** and you will be able to print a **paper copy** or create a **PDF** of the report.



If you experience any problems using your Mobil-O-Graph ABPM, please contact our technical support team on 0114 399 0010 or visit: www.numed.co.uk/support





Alliance House, Roman Ridge Road, Sheffield S9 1GB
Tel: 0114 243 3896 Fax: 0114 243 3413 info@numed.co.uk
www.numed.co.uk